

## Accepted Scanner Manufacturers

We proudly accept STL files from all intraoral scanner manufacturers

Our email address for all systems is: [emanuel.khaimov@gmail.com](mailto:emanuel.khaimov@gmail.com). You can usually search for our email within your scanner software to find our lab.

If you have any further questions, please call us at [516-792-5882](tel:516-792-5882) or [206-948-4812](tel:206-948-4812). Instructions for specific systems, please see below.

### TRIOS/3-Shape

- Go to [us.3shapecommunicate.com](http://us.3shapecommunicate.com) in a web browser
- Click connections, then click add connection and search for [emanuel.khaimov@gmail.com](mailto:emanuel.khaimov@gmail.com), select our lab and press connect
- After connecting to us as a Lab, select Emanuel Dental Lab when sending files

### Medit

- Sign into your MeditLink.com account
- Search for [emanuel.khaimov@gmail.com](mailto:emanuel.khaimov@gmail.com)
- Add Emanuel Dental Lab

### ITERO®

- Call [800-577-8767](tel:800-577-8767)
- Select Option 1
- Request that Emanuel Dental Lab is added to your scanner and Identify our lab using our Account ID #: [188142](#)
- After Emanuel Dental Lab has been added, restart your scanner
- After connecting to us as a lab, select Emanuel Dental Lab on your scanner when sending files

### CEREC

- Create or Login to your CEREC Connect account
- Select "MY CEREC CONNECT"

- Type "Emanuel Dental Lab" in the box for Company Name
- Click FIND
- Check the box to the bottom left for "Emanuel Dental Lab"
- Click ADD to complete the registration

## 3M™ TRUE DEFINITION SCANNER

- Call 3M™ support at [800-634-2249](tel:800-634-2249) select Option 3
- Select Option 1 and request support Emanuel Dental Lab be added to your scanner
- 3M™ will then confirm with Emanuel Dental Lab and add connection remotely

## CARESTREAM

- Visit Carestream Connect on your scanner
- Search for Emanuel Dental Lab
- Add Emanuel Dental Lab
- Select Emanuel Dental Lab when submitting scans raw STL files
- Email Emanuel Dental Lab at [emanuel.khaimov@gmail.com](mailto:emanuel.khaimov@gmail.com)

## E4D-PLANMECA PLANSCAN

- Select Find a lab option on your scanner
- Search for either Emanuel Dental Lab or [emanuel.khaimov@gmail.com](mailto:emanuel.khaimov@gmail.com)
- Add Emanuel Dental Lab
- Select Emanuel Dental Lab when submitting scans